Feelvita Acticur

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Shenzhen Pango Medical Electronics Co., Ltd

Main Site: Building 2, No. 25 Fenghuang Raad, Industrial Zone, Xikeng First Village, Henggang Street, Langgang District, Shenzhen, 518115 Guangdong, P. R. China. Additional site 1: 2-4 Floor ,No.5 Shanzhuang Rd., Xikeng Village, Henggang Town, Langgang District, Shenzhen, 518115 Guangdong, P. R. China.

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+49 (0) 6431 9 21 96 00

Content

Product components	37
General information	39
Explanation of symbols	39
Safety	40
Proper use	40
Safety instructions	40
Contraindications	40
General warnings	41
Example if signs	45
Before first use	48
Use	51
Cleaning and care	55
Interesting facts about blood pressure	55
Troubleshooting	57
Appendix	59
Disposal	68
Technical data	68

Product components

- 1 Blood pressure monitor
- 2 Storage box
- 3 Memory button ("MEM" button)
- 4 On/off button ("ON/OFF" button)
- 5 SET button
- 6 Battery compartment cover
- 7 Systolic pressure
- 8 Diastolic pressure
- 9 Unit of pressure measurement
- 10 Measured pulse value
- 11 "Pulse" symbol

PLEASE TAKE THE TIME TO READ THESE INSTRUCTIONS FOR USE CAREFULLY.

FOR YOUR OWN PROTECTION AND THE PROTECTION OF THIRD PARTIES, PAY PARTICULAR ATTENTION TO THE CHAPTER ON SAFETY INSTRUCTIONS AND THE WARNINGS CONTAINED IN THE DOCUMENT. KEEP THESE INSTRUCTIONS FOR USE FOR FUTURE REFERENCE. ONLY PASS THE DEVICE ON TO THIRD PARTIES TOGETHER WITH THE OPERATING INSTRUCTIONS.

THE ILLUSTRATIONS IN THESE INSTRUCTIONS FOR USE ARE INTENDED AS A SCHEMATIC REPRESENTATION OF THE CORRECT APPLICATION. THEY MAY DIFFER IN DETAIL FROM THE ORIGINAL PRODUCT.

All available accessories for the appliance are included in these operating instructions. The scope of delivery of the set you have purchased can be found on the packaging or on the separate insert. Parts not included in your set can of course be purchased at a later date.

General information

Explanation of symbols

The following symbols and signal words are used in this manual, on the device or on the packaging.

WARNING!

Failure to observe this warning could lead to a serious or potentially lethal accident.

CAUTION!

Failure to observe this caution could lead to injury.

NOTE!

Information about potential material damage.

Safety

Proper use

- The blood pressure monitor is designed for fully automatic, non-invasive measurement of arterial blood pressure and pulse values on the wrist and can be used on wrists with a circumference of 13.5–19.5 cm. It displays the diastolic and systolic blood pressure as well as the pulse rate.
- The device is intended for private use by adults indoors only. It is not suitable for commercial or industrial use. The intended user is the patient.
- The device must not be used for disease assessment, first aid or continuous measurement monitoring.
- The patient must not carry out self-assessment or self-treatment based on the measurement results, but must consult a doctor for this purpose.
- Only use the device as described in the instructions. Any other use is considered improper.
- Maintenance and servicing must not be carried out while the ME device is in use. The user can maintain the product. The maintenance method is described in the instructions for use.

Safety instructions

Contraindications

- Do not use the blood pressure monitor on newborns, children or pets.
- Persons with limited physical, sensory or mental abilities should be supervised by a person responsible for their safety and instructed on how to use the device.

- Contact a doctor before using the device if any of the following apply to you:
 - Heart rhythm disorders
 - Circulatory disorders
 - Diabetes
 - Pregnancy / pre-eclampsia
 - Hypotension
 - Chills / tremors
 - Pacemaker or other electrical implants
- Do not use together with a high-frequency surgical device
- Do not place the cuff over wounds, as this may cause further injury
- Ensure that the cuff is not applied to a wrist where arteries or veins are undergoing medical treatment (e.g. intravascular access)

General warnings

- The following circumstances may lead to measurement errors or malfunctions or negatively affect the accuracy of the measurements:
 - Use of the device outside the home or under the influence of movement
 - Diseases of the cardiovascular system

- Use of the device at the same time as other medical electrical devices
- Use of the device outside the specified storage and operating conditions
- During inflation of the cuff, the function of the respective wrist/arm may be impaired.
- Do not take measurements more often than necessary.
- Do not restrict blood circulation for longer than necessary. If the device does not work properly, remove the cuff from your wrist.
- Place the cuff only on the wrist and not on other parts of the body.
- Do not take more than three measurements in a row. Allow at least 5 minutes between measurements.
- Do not measure your blood pressure more than six times a day.

NOTE!

Risk of damage to materials, property and equipment

- The accuracy of the measured values and the service life of the device depend on careful

handling. Protect the device from impact, moisture, dirt, extreme temperature fluctuations and direct sunlight.

- Allow the device to reach room temperature before taking measurements.
- Do not drop the device.
- Always store the device in the box provided when not in use.
- Keep the device away from strong electromagnetic fields, radio equipment and mobile phones.
- Remove the batteries if you do not intend to use the device for a long period of time.
- Observe the value on the LCD display. The pressure must not exceed 280 mmHg during measurement. If it does, press the on/off button to stop the measurement.
- Do not bend the wrist cuff forcefully.
- Only use the accessories listed in the instruction manual. The use of parts not approved by the manufacturer may result in malfunctions or injury. For service information, spare parts lists, etc., please contact your dealer.

Handling batteries:

- Batteries can be life-threatening if swallowed. If a battery is swallowed, seek medical attention immediately.
- If a battery has leaked, avoid contact with skin, eyes and mucous membranes. In case of contact with battery acid, rinse the affected areas immediately with plenty of clean water and seek medical attention immediately. Clean the battery compartment with a dry cloth.
- Explosion hazard – Do not throw batteries into fire, open, disassemble or crush them.
- Ensure correct polarity (+/-) when inserting.
- Only use the same or equivalent battery types.
- Replace all batteries at the same time.
- Do not use rechargeable batteries.

Examples of signs

*The warning signs and sample icons shown here are listed for your safe and correct use of the unit, so as to prevent injuries or damages to the device.

*The icons and meanings are as follow.

The Ø icon indicates prohibitions (what you should not do). Malters involving actual prohibitions are indicated by text or pictures in or near (Ø). The left icon refers to „general prohibition“.

The ! icon indicates something that is compulsory (what must always be observed). Malters involving actual compulsory actions are indicated by text or pictures in or near !. The left icon refers to „general compulsion“.

The ⓧ icon indicates something can't be disassembled or „Don't disassemble“ Malters involving actual compulsory actions are indicated by text or pictures in or near ⓧ. The left icon refers to „general prohibition“.

Type BF Applied part

IP Classification: IP20

Caution

Please refer to the instructions
for use

Consult instruction for use

Indicates a medical device that needs to be protected from moisture

Indicates medical device

Contact its local authorities to determine the proper method of disposal of potentially bio hazardous parts and accessories.

The following symbol indicates that the device is MR-unsafe:

MR Unsafe

Do not use this equipment in the MRI scan room

Patient must follow doctor's instruction and should not perform self-judgment and self-treatment by the measuring result, Self-diagnosis of measured results and treatment are dangerous, The device should not be used to judge illness, first aid and continuously monitor measuring. This device can not be used for Patient transport and surgical care .It can be used in household or fixed places only. Please press „on/off„ button to stop work when you feel uncomfortable with the wrist, or if the air is inflating abnormally without stop.	 Caution 
This device should not be used by children under 18 years old or people who cannot express their will, otherwise it will cause harm. Do not use the unit for purpose other than measuring blood pressure. May cause accident or trouble. Please do not use mobile phone around the device. Please do not use the device around the magnetic field. The device is prohibited from being used during movement. Do not use the equipment in outdoor or shower rooms.	

Do not disassemble, repair, or remodel the main unit or the wrist cuff of the blood pressure monitor.
Will cause the unit to function erroneously.

Before first use

CAUTION!

Risk of suffocation from plastic film and bags! Keep children and animals away from the packaging material.

- **Check** the set for completeness and any transport damage. If you have any complaints, please contact our customer service.
- **Remove** the packaging material and any labels on the product that are used for transport protection or advertising.
- If you are taking medication, consult your doctor to determine the best time to measure your blood pressure. Never change a prescribed medication without first consulting your doctor.
- In people with irregular or unstable peripheral blood circulation due to diabetes, liver disease, arteriosclerosis, etc., blood pressure values measured (by a doctor) on the upper arm may vary compared to those measured on the wrist.

Inserting the batteries

1. Open the battery compartment cover.
2. Insert 2 AAA batteries (1.5V) into the device (batteries not included). Ensure that the polarity (+/-) is correct. Use only LR03 AAA batteries.
3. Close the battery compartment cover.

NOTE:

If the „BATTERY LOW“ indicator flashes on the display, replace all batteries with new ones. Do not use old and new batteries at the same time. This may shorten the battery life or cause the device to malfunction.

The „BATTERY LOW“ indicator does not appear when the batteries are completely empty.

Battery life varies depending on the ambient temperature and may be shorter at low temperatures.


The batteries may leak and cause malfunctions. If you do not use the device for a long period of time, remove the batteries as a precaution.

Only use the specified batteries.

Setting the time


1. With the device switched off, press and hold the „SET“ button until the year appears on the display.
2. Press the „MEM“ button repeatedly until the correct year appears on the display. Pressing „SET“ changes to the month.
3. Pressing or holding the „MEM“ button repeatedly changes the flashing display, pressing „SET“ confirms the respective value. Use this to set the correct data step by step.

4. Display format: Month-Day Hour:Minute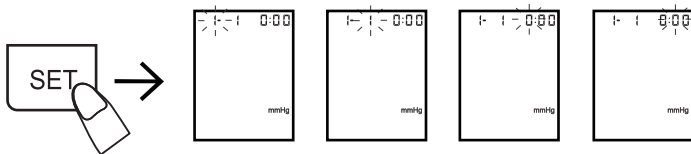

Setting the display unit – mmHg/kPa

- The device can display blood pressure in mmHg or kPa. The display is preset to mmHG at the factory.
- To change the setting, press and hold the „ON/OFF“ button for 10 seconds until the display shown below appears. Use „MEM“ to select the desired display. Press the „ON/OFF“ button to save and exit the display.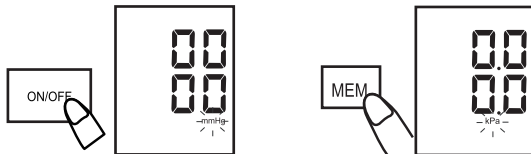

Use

Adopt the correct posture

- Sit comfortably at a table. Place your wrist on the table.
- Relax for about 5 to 10 minutes before taking the measurement.
- Raise your hand so that the wrist cuff is at heart level.
- Remain still during the measurement and avoid talking.
- Do not take a measurement immediately after physical activity or a bath.
- Measure your blood pressure at approximately the same time every day.

Fitting the wrist cuff

- In principle, blood pressure can be measured on both wrists. Certain differences between the left and right sides are normal and physiologically determined. Always measure your blood pressure on the same wrist.
- Only use the device with the cuff that is permanently attached to it when delivered. Before use, make sure that your wrist circumference is within the printed range (13.5–19.5 cm).
- Expose your wrist. Make sure that your arm is not constricted by tight clothing or jewellery.
- Place the wrist cuff so that the palm of your hand and the display are facing upwards.
- Leave a gap of about 1–2 cm between the heel of your hand and the cuff.
- Close the cuff tightly using the Velcro fastener.

Measuring blood pressure

1. Put on the wrist cuff as described in „Fitting the wrist cuff“.
2. Press the „ON/OFF“ button. All symbols will appear on the display for two seconds. The display will then automatically switch to measurement mode and show „0“ or the last measurement value.


3. The measurement starts automatically. The cuff is inflated for this purpose. Once the cuff is fully inflated, the pressure is slowly reduced and the measurement is taken. The pulse indicator flashes during the measurement. Once the measurement is complete, the results are displayed.

Readout memory


- The device stores up to 90 readings in its internal memory.
- To access them, press the „MEM“ button.
- When the display first appears, „RU9“ appears in the upper right corner of the display. The values show the average of the last 3 measurements (regardless of the time period).
- Pressing the „MEM“ or „SET“ button repeatedly displays the last measured values including the date/time of the respective measurement. „MEM“ starts backwards from the last measurement, „SET“ starts forwards from the first stored measurement.
- To clear the memory, press and hold the „MEM“ button for five seconds. The memory is cleared and „NO“ appears in the lower right corner of the display.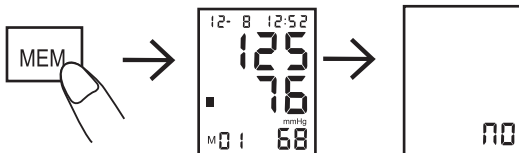


Display of WHO blood pressure classification

The WHO (World Health Organisation) has established an internationally recognised classification for the assessment of measured blood pressure values.

Reference Material: Journal of Hypertension
1999, Vol 17 No.2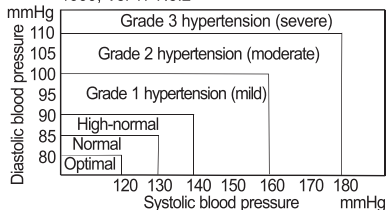

A classification of the measured values according to the WHO blood pressure classification is shown on the left side of the display.

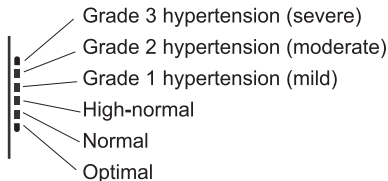The classification is independent of age, gender and other characteristics and is only intended as a rough guide. Only a doctor is able to assess your individual blood pressure values and derive medical indications from them.

Cleaning and care

WARNING!

Risk of electric shock

Never hold the device under running water and do not immerse it in water or other liquids.

- Clean the device by wiping it with a damp cloth.
- Do not use sharp or abrasive cleaning agents or cleaning pads.
- Store the device in the supplied case in a dry, cool place away from direct sunlight and out of the reach of children and animals.
- Allow all parts to dry completely before storing.

Interesting facts about blood pressure

1. What is blood pressure?

Blood pressure is the force exerted by the blood on the walls of the arteries. Systolic pressure occurs when the heart contracts. Diastolic pressure occurs when the heart expands.

Blood pressure is measured in millimetres of mercury (mmHg). Natural blood pressure is represented by the baseline pressure, which is measured in the morning at rest and before eating.

2. What is high blood pressure and how is it controlled?

High blood pressure, i.e. abnormally high arterial blood pressure, can cause many health problems if left untreated, including stroke and heart attack. High blood pressure can be controlled by lifestyle changes, stress avoidance and medication under medical supervision.

To prevent high blood pressure or keep it under control:

- Do not smoke
- Reduce your salt and sugar intake
- Maintain a healthy weight
- Exercise regularly
- Go for regular check-ups

3. Blood pressure fluctuations

A person's blood pressure fluctuates greatly on a daily and seasonal basis. It can vary by 30 to 50 mmHg due to different conditions throughout the day.

These fluctuations are even more pronounced in people with high blood pressure. Blood pressure normally rises during work or leisure activities and falls to its lowest level during sleep. Therefore, do not worry too much about the results of a single measurement.

Measure your blood pressure at the same time every day as described in these instructions and determine your normal value. Many measurements provide a more comprehensive blood pressure profile. Be sure to note the date and time of your blood pressure measurement.

Consult your doctor to interpret your blood pressure data.

Troubleshooting

Error display	Possible cause	Solution
No display when pressing the "ON/OFF" button or battery symbol flashes	No batteries inserted.	Insert batteries.
	Charge level insufficient.	Insert new batteries.
	Batteries have been inserted with the wrong polarity.	Insert batteries with correct polarity.
E1: Pressure cannot be increased normally	Air leak in the cuff.	Check the cuff and replace if necessary.
E3: Inflation pressure too high	Pressure value exceeds 299 mmHg.	Measure again or send the device to the retailer for calibration.
E2/E4: Movement during measurement	Hand or body movement during measurement.	Remain still and adopt the correct position, then measure again.

Battery symbol is lit	Battery low.	Insert new batteries and measure again.
Systolic or diastolic value too high	1. Cuff was below heart level 2. The cuff was not positioned correctly 3. Movement or talking during measurement	Ensure correct position and application, then measure again. Remain still during measurement
Systolic or diastolic value too low	1. Cuff was above heart level 2. Movement or talking during measurement	

CAUTION

If the problem persists, please contact our customer service. Never open the device!

Appendix

Guidance and manufacturer's declaration - electromagnetic emissions

The Model PG-800A31 Series Electronic Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the Model PG-800A31 Series Electronic Blood Pressure Monitor should assure that it is used in such an environment.

Emissions	Compliance	Electromagnetic environment-guidance
RF emissions CISPR 11	Group 1	The Model PG-800A31 Series Electronic Blood Pressure Monitor uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.

RF emission CISPR 11	Class B	The Model PG-800A31 Series Electronic Blood Pressure Monitor is used in harne and it's powered by DC 3V.
Harmonie emissions IEC 61000-3-2	N/A	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	N/A	

Guidance and manufacturer's declaration - electromagnetic immunity

The Model PG-800A31 Series Electronic Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the Model PG-800A31 Series Electronic Blood Pressure Monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Electrostatic discharge (ESD) IEC 61000-4-2	+8kV contact +2kV, + 4kV. +8kV, + 15kV air	+8kV contact +2kV, + 4kV. +8kV, + 15kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.

Power frequency (50Hz/60Hz) magnetic field IEC 61000-4-8	30 A/m, 50/60Hz	30 A/m, 50/60Hz	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.
NOTE: UT is the a.c. mains voltage prior to application of the test level.			

Guidance and manufacturer's declaration - electromagnetic immunity

The Model PG-800A31 Series Electronic Blood Pressure Monitor is intended for use in the electromagnetic environment specified below. The customer or the user of the Model PG-800A31 Series Electronic Blood Pressure Monitor should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz 6 Vrms 150 kHz to 80 MHz outside ISM bands	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the Model PG-800A31 Series Electronic Blood Pressure Monitor, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Recommended separation distance $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$

Radiated RF IEC 61000-4-3	10V/m 80 MHz to 2.7 GHz	10V/m	$d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$ 80MHz to 800MHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$ 800MHz to 2.7GHz where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres(m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey, ^a should be less than the compliance level in each frequency range ^b. Interference may occur in the vicinity of equipment marked with the following symbol: 
NOTE 1: At 80 MHz and 800 MHz, the higher frequency range applies.			

NOTE 2:

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

a The ISM (industrial, scientific and medical) bands between 0,15 MHz and 80 MHz are 6,765 MHz to 6,795 MHz; 13,553 MHz to 13,567 MHz; 26,957 MHz to 27,283 MHz; and 40,66 MHz to 40,70 MHz. The amateur radio bands between 0,15 MHz and 80 MHz are 1,8 MHz to 2,0 MHz, 3,5 MHz to 4,0 MHz, 5,3 MHz to 5,4 MHz, 7 MHz to 7,3 MHz, 10,1 MHz to 10,15 MHz, 14 MHz to 14,2 MHz, 18,07 MHz to 18,17 MHz, 21,0 MHz to 21,4 MHz, 24,89 MHz to 24,99 MHz, 28,0 MHz to 29,7 MHz and 50,0 MHz to 54,0 MHz.

b The compliance levels in the ISM frequency bands between 150 kHz and 80 MHz and in the frequency range 80 MHz to 2,7 GHz are intended to decrease the likelihood that mobile/portable communications equipment could cause interference if it is inadvertently brought into patient areas. For this reason, an additional factor of 10/3 has been incorporated into the formulae used in calculating the recommended separation distance for transmitters in these frequency ranges.

c Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the Model PG-800A31 Series Electronic Blood Pressure Monitor is used exceeds the applicable RF compliance

normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the Model PG-800A31 Series Electronic Blood Pressure Monitor.

d Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

Recommended separation distances between portable and mobile RF communications equipment and the Model PG-800A31 Series Electronic Blood Pressure Monitor

The Model PG-800A31 Series Electronic Blood Pressure Monitor is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the Model PG-800A31 Series Electronic Blood Pressure Monitor can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the Model PG-800A31 Series Electronic Blood Pressure Monitor as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter (W)	Separation distance according to frequency of transmitter (m)		
	150 KHz to 80 MHz $d = \left[\frac{3.5}{V_1} \right] \sqrt{P}$	80 MHz to 800 MHz $d = \left[\frac{3.5}{E_1} \right] \sqrt{P}$	800 MHz to 2.7 GHz $d = \left[\frac{7}{E_1} \right] \sqrt{P}$

0.01	0.12	0.12	0.23
0.1	0.38	0.38	0.73
1	1.2	1.2	2.3
10	3.8	3.8	7.3
100	12	12	23

For transmitters rated at a maximum output power not listed above the recommended separation distance d in metres (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1:

At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.

NOTE 2:

These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

Shenzhen Pango Medical Electronics Co., Ltd

Main Site: Building 2, No. 25 Fenghuang Raad, Industrial Zone, Xikeng First Village, Henggang Street, Langgang District, Shenzhen, 518115 Guangdong, P. R. China. Additional site 1: 2-4 Floor ,No.5 Shanzhuang Rd., Xikeng Village,Henggang Town, Langgang District, Shenzhen, 518115 Guangdong, P. R. China.

Tel: +86-755-33825988 Fax: +86-755-33825989

EU REP

Lotus NL B.V.

Address: Koningin Julianaplein 10, 1e Verd, 2595AA, The Hague, Netherlands.

Tel: +31644168999

Email: info@lotusnl.com

Disposal

Packaging

The packaging material is recyclable. Dispose of the packaging in an environmentally friendly manner and take it to the recycling centre.

Electrical appliances

The crossed-out wheellie bin symbol indicates that this product must not be disposed of with normal household waste. Please dispose of electrical and electronic devices via your local take-back system. For more information on the disposal and recycling of this appliance, please contact your local waste disposal service, the shop where you purchased the product or the manufacturer.

Technical data

DEVICE	
Model	PG-800A31
Measurement	Oscillometric, non-invasive measurement at the wrist
Measuring range	Pressure: (30~280) mmHg Pulse: (40~199) beats/minute
Display accuracy	Static pressure: +/- 3 mmHG Pulse: +/- 5%

Memory	90 measurements
Dimensions	Approx. 6.2 x 7.8 x 3.1 cm
Weight	Approx. 130 g (without batteries)
Cuff size	13.5–19.5 cm
Power	2x 1.5V batteries (LR03 or AAA)
Permissible operating environment	+5°C to +40°C; relative humidity max. 90%; ambient pressure 700-1060 hPa
Permissible storage conditions	-20°C to 55°C; relative humidity max. 90%
Operating time	max. 80 minutes
Classification	Internal power supply, no AP or APG, continuous operation, type BF

Customer service

If you have any questions about the device or spare parts/accessories, please contact your local retailer.

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