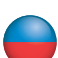

CLEVER CANE

-  Innovative! Two grip design for added support.
-  Innovant! Design à double poignée pour plus de soutien.
-  Innovatief! Twee handvatten voor optimale steun en zekerheid.
-  Innovativo design a doppia presa per un maggiore sostegno.
-  Innovatives Dual-Griff-Design für zusätzliche Unterstützung.
-  Инновационный дизайн с двумя захватами, для создания лучшей опоры.
-  Diseño de agarre doble innovador para un mayor soporte.
-  Innovativ dubbelfattning med stödjande design.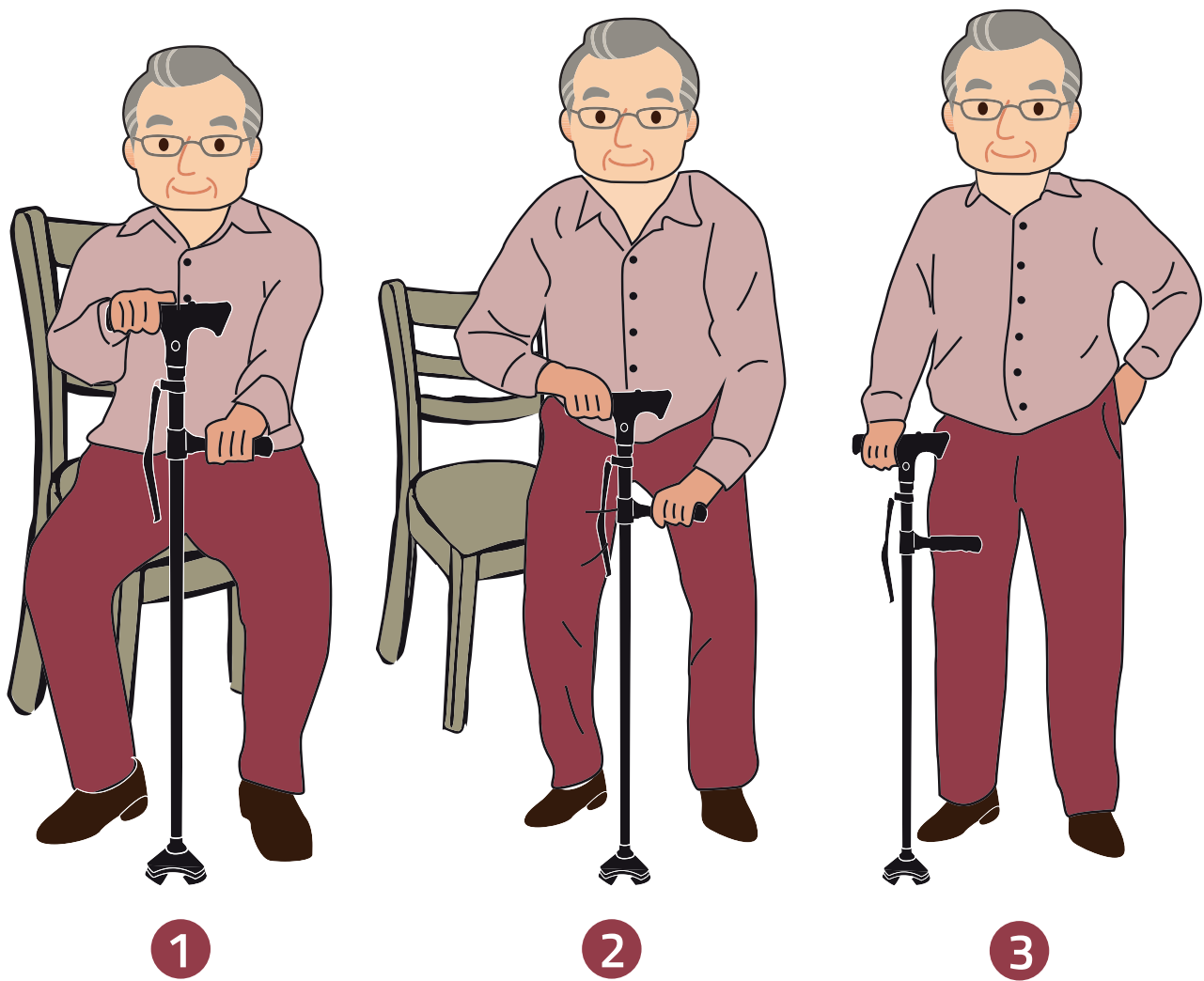

Dual handle usage:

1. Spread handles

2. Lean forward

3. Push yourself up

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2. Penchez vous

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2. Leun voorover

3. Druk uzelf op.

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1. Stendere le maniglie

2. Piegarsi in avanti

3. Alzarsi

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1. Griffe ausklappen

2. Vorbeugen

3. Hochdrücken

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1. Extienda el mango

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CLEVER CANE

Innovative! Two grip design for added support.

 INSTRUCTIONS FOR USE

Item 02100 – Made in China

Product description

1. Handle ■ 2. Cane sections ■ 3. Wrist strap ■ 4. Elastic cord ■ 5. Length adjustment button ■ 6. LED-light ■ 7. Light switch ■ 8. Battery compartment ■ 9. Alarm ■ 10. Alarm switch ■ **Batteries not included.**

Unfolding Clever Cane

Clever Cane has a wrist strap that keeps the sections of the cane together when the cane is folded up. To unfold the cane, first unfasten the wrist strap (3). Grasp the handle (1), hold it up and let the sections of the cane (2) unfold by the force of gravity.

IMPORTANT: To prevent falls, always ensure that all of the cane sections (2) are perfectly aligned and locked in place (see Figure B) and the cane is stiff.

CAUTION: Uncontrolled release of the elastic cord (4) inside the cane can cause severe injury to unprotected body parts, particularly the eyes. If the elastic cord (4) breaks, becomes damaged or wears out, stop using the product. Do not stretch the elastic cord (4) more than is necessary to unfold or fold up the cane (2).

Adjusting the length to suit your needs

There are five different length settings. To make it longer, hold down the length adjustment button (5) as shown in Figure C, pull up on the handle and release the adjustment button to let it lock into a lower slot. To make the cane shorter, hold down the length adjustment button (5), push down on the handle and release the adjustment button to let it lock into a higher slot.

Using the LED-light

An adjustable light (6) is located at the end of the handle (1). To turn the light on and off, press the light switch (7) on the cane handle (1). The light pivots up and down so you can point it in the direction you wish.

Using the alarm in emergency situations

To turn the alarm on and off, press the alarm switch (10) located at the head part of the cane handle.

Folding up Clever Cane

To store the cane, separate the sections as shown in Figure D and group them together. Fasten them with the wrist strap (3). Secure the sections with the fasteners at each end of the strap.

Insert and change batteries

Clever Cane operates with 2 AAA batteries (not included). The battery compartment (8) is located at the end of the handle, open it and insert or change batteries.

Safety instructions:

- Read these manual before using Clever Cane for the first time.
- Always consult your doctor before using Clever Cane.
- Do not disassemble the body (with elastic cord) of the cane.
- The product is not for medical use.
- Not suitable for skiing.
- Never use the cane on ice, wet or slippery surfaces.
- This product does not aim to be a walking aid as per EN 1985 (on hiking sticks).
- Do not use the cane if it is defective.
- Not intended for persons over 115kg.

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Your Products B.V.